


iBreathe HF60 Quick Guide



iBreathe HF60 is a heated & humidified High flow Oxygen therapy system.



Front View

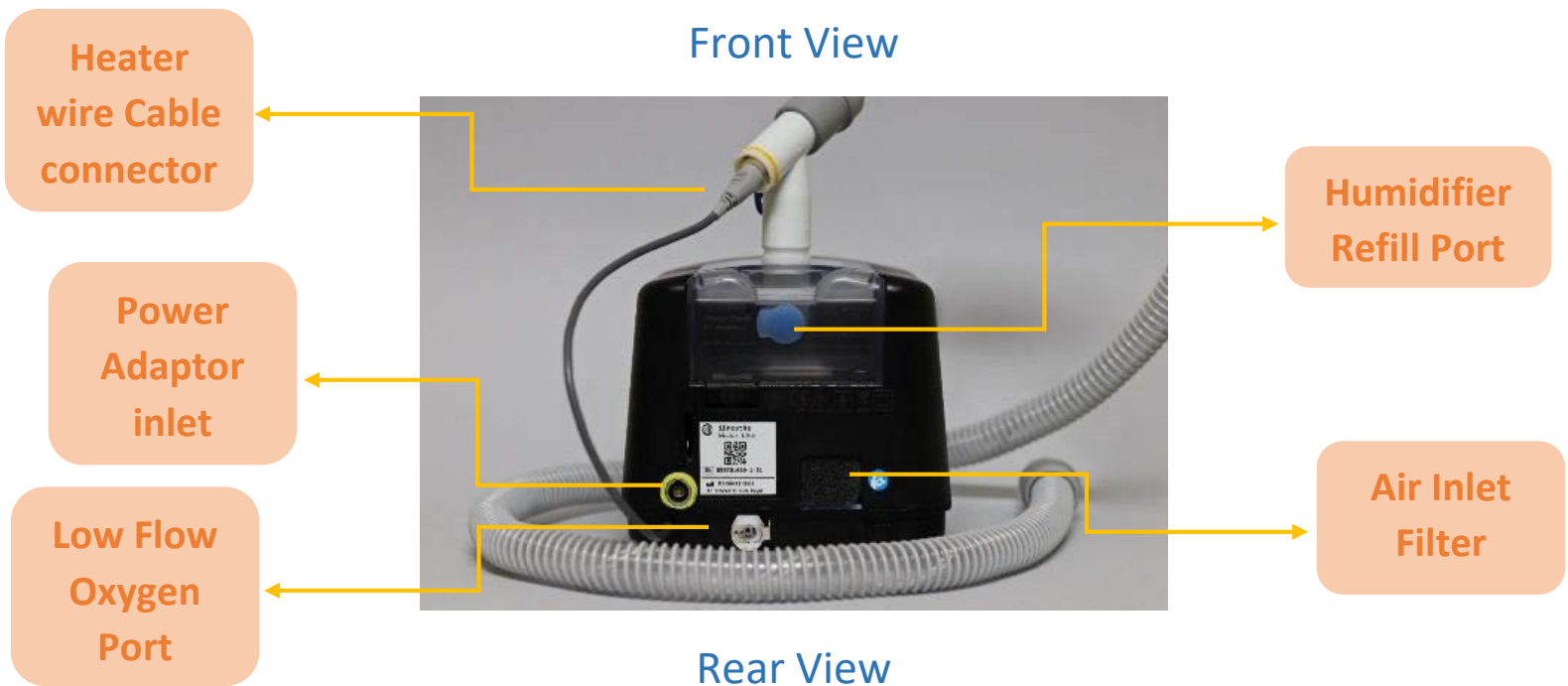

Rear View




Attention: This Quick Guide doesn't replace the User Manual.

Attention: Always monitor closely the water level at the humidifier chamber as this may affect the therapeutic humidification level required for the patient and may cause discomfort, damage of upper track mucosa, and/or equipment failure.

Attention: Change Air Inlet filter for each patient.

Attention: Don't connect humidified oxygen source (make sure the oxygen flowmeter water tank is empty).



iBreathe HF60 Quick Guide

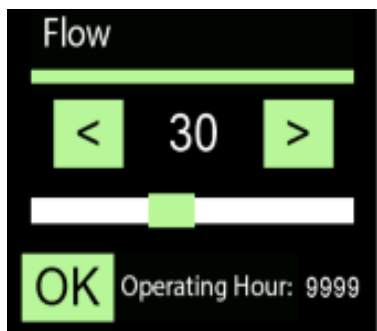

Flow Range from 2-60

Green Indicates the therapy is OFF

Red indicates ongoing therapy

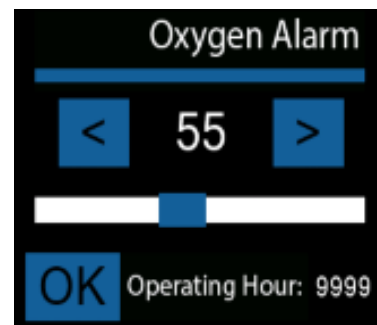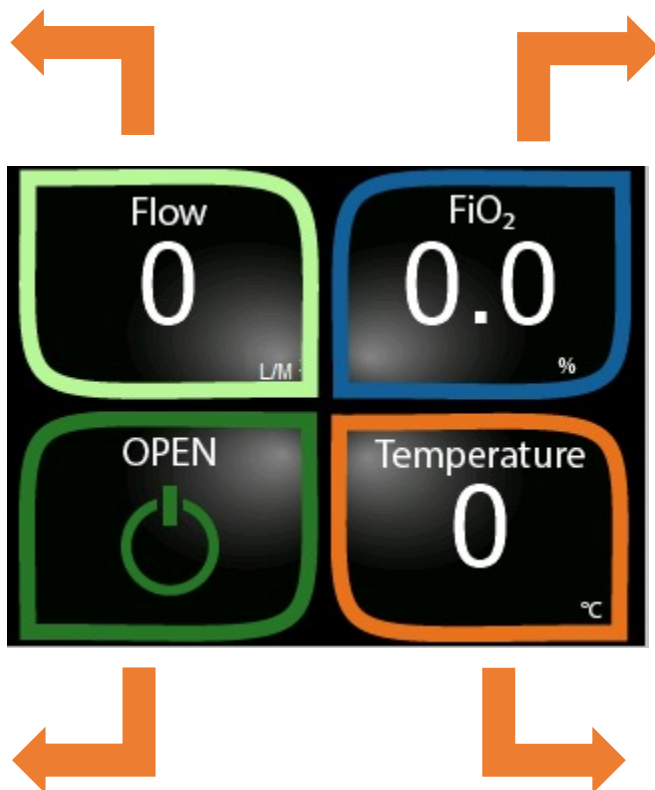


Oxygen Level alarm from 20 to 100%



Temperature Range from 30 to 38° C


Alarms:

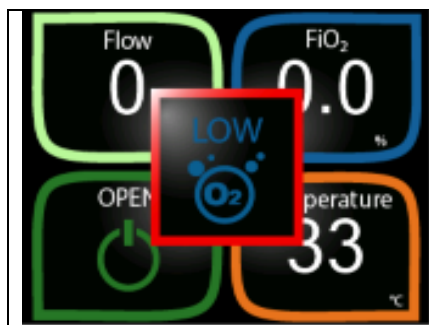

Low Oxygen



Low Temperature

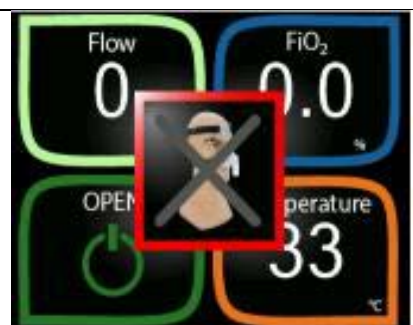

Disconnection



Attention: Avoid liquid spill on the unit, use a Syringe to refill the humidification chamber from the port located at the rear side. **Make sure to close the port after the refill.**

